

MEGAMAT 5 (MFZ)

CARE, HOSP

Installation Instructions

(Translation of the original installation instructions)

Foreword

Document revision history

Version	Date	Modification, change
(-)	04/11	First release
(a)	07/12	Edition II
(b)	11/12	LR / lifter release
(c)	03/13	GQR
(d)	09/13	Installation of LR / lifter release
6.0	09/2019	Installation, ratings plate, declaration of conformity and installation

Disclaimer and exclusion of liability

DewertOkin is not responsible for damage resulting from:

- failure to observe these instructions,
- changes made to this product which have not been approved by DewertOkin, or
- the use of replacement parts which have not been approved or manufactured by DewertOkin.

Manufacturer's address

DewertOkin GmbH
 Weststrasse 1
 32278 Kirchlegern, Germany
 Germany
 Tel: +49 (0)5223/979-0
 Fax.: +49 (0)5223/75182
<http://www.dewertokin.de>
Info@dewertokin.de

Creation of a complete operating instruction manual for the entire end product

These instructions are only intended to be used by the end-product manufacturer. They should not be given to the operator of the end product. The factual information contained within may be used as a basis when creating the end-product manual.

The warning and danger notices are best suited for use in the end product's manual. However it is not sufficient to simply follow these notices. You should also carry out an internal risk assessment for your end product. This can then be used as the basis for the safety notices in your manual.

These installation instructions do not contain all information required to safely operate the end product. They only describe the installation and operation of the drive as partially completed machinery.

The instructions are intended for the technicians responsible for manufacturing an end product and not for the operators of the end product.

Notice for customers in EU nations**German Inspection Authority (TÜV SÜD Product Service) testing label**

The construction of the drive has been inspected by the German TÜV SÜD Product Service Inspection Authority. TÜV SÜD Product Service also monitors the production of the MEGAMAT MFZ. The official German TÜV SÜD Product Service certifies this construction inspection and production monitoring.

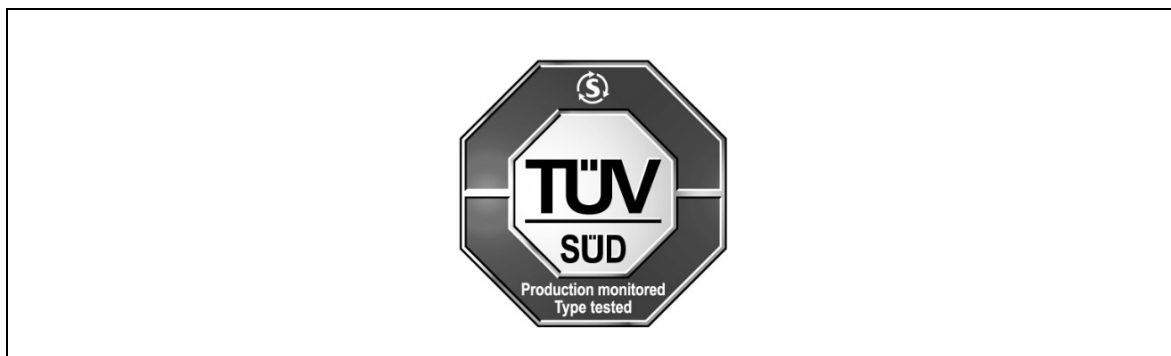


Figure 1 TÜV SÜD Product Service Safety Mark

Table of Contents

Foreword	3
Document revision history	3
Disclaimer and exclusion of liability	3
Manufacturer's address	3
Creation of a complete operating instruction manual for the entire end product	3
Notice for customers in EU nations	4
Table of Contents	5
1. General Information	7
1.1 Configurations	7
1.2 About these installation instructions	7
1.3 Availability of this document	7
1.4 Conventions used	8
2. Safety notices	9
2.1 Proper and intended usage	9
2.2 Safety notices within the installation instruction and the operating instructions for the entire machine	10
2.3 Selection and qualification of personnel	10
2.4 Notice on safety during operations	11
2.5 Product identification	12
3. Possible combinations	14
4. Description	15
4.1 Components	15
5. Technical specifications	21
6. Installation	24
6.1 Safety notices to observe during installation	24
6.2 Installation procedure	26
7. Operating notes	37
7.1 General information	37
7.2 Notice for operating with optional configuration	38
8. Troubleshooting	42

9.	Maintenance	44
9.1	Maintenance	44
9.2	Cleaning and care	45
10.	Disposal	46
10.1	Packaging material	46
10.2	Drive components	46
Declaration of incorporation/installation		47
EU Declaration of Conformity		48
Additional information		49

1. General Information

1.1 Configurations



The MEGAMAT MFZ drive can be used in several different configurations. The term MEGAMAT MFZ also refers to the following facility versions:

- MEGAMAT MFZ
- MEGAMAT MF1
- MEGAMAT MF2

1.2 About these installation instructions

These installation instructions must be followed closely in order to install this drive successfully and safely in the end product. These instructions are not an operating manual for the end product.

These instructions will help you to minimize danger, repair costs and down times. They will also help you to maximize the reliability and lifespan of the end product.

	 CAUTION
	<p>The notices in these instructions must be followed! Following the guidelines during installation and connection procedures will help to minimize:</p> <ul style="list-style-type: none">• the risk of accident and injury, and• damage to the drive system or the end product.

These installation instructions have been written with due care and attention. However, we cannot guarantee that the data, images and drawings are complete and correct nor do we accept any liability for the information contained therein, unless required by law.

- We reserve the right to make unannounced technical changes in the course of our continual product improvement process!

1.3 Availability of this document






As manufacturer of the end product, you are obligated to comply with Machinery Directive 2006/42/EC. This directive stipulates that the installation instructions must be kept on file for governmental inspection purposes.

1.4 Conventions used

Notices which do not relate to safety are indicated in these instructions with a triangle:

- Triangular notice symbol

Safety notice explanations



	<div data-bbox="395 629 1444 696"> WARNING</div> <div data-bbox="395 696 1444 781">WARNING indicates a hazardous situation which, if not avoided, <i>could</i> result in death or serious injury.</div>
	<div data-bbox="395 833 1444 900"> CAUTION</div> <div data-bbox="395 900 1444 983">CAUTION indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.</div>
	<div data-bbox="395 1034 1444 1079">NOTICE</div> <div data-bbox="395 1079 1444 1158">NOTICE is used to address practices which are not related to personal injury but may result in damage to the product or surroundings.</div>

2. Safety notices

2.1 Proper and intended usage



The MEGAMAT MFZ drive is meant to be installed in an end product.



- It provides motor adjustment capabilities for movable furniture. It should be used in conjunction with suitable fittings and mechanics.
- It can be used for care purposes (CARE).
- It can be used in a hospital (HOSP).
- It can be used for raising and lowering persons (as a patient lifter).

	 CAUTION
	This drive should only be used for the applications described above. Any other use is forbidden. Improper usage can lead to accidents or destruction of the unit. Such non-approved applications will lead immediately to the expiration of all guarantee and warranty claims on the part of the end-product manufacturer against the manufacturer.



Improper usage

Be sure to follow the notices below concerning improper usage. You should include them in your product manual in order to inform the users of your end product.



	 WARNING
	The MEGAMAT MFZ drive should not be used: <ul style="list-style-type: none"> • in any environment where combustible or explosive gases or vapours (e.g., anaesthesiology) may be present, • with pulling or pushing forces that exceed those specified on the ratings plate, • in a moist environment, • outdoors, • in any application that will be cleaned with an automated washing system.

	 CAUTION
	The MEGAMAT MFZ drive may not be operated: <ul style="list-style-type: none"> • by small children, • by frail or infirm persons without supervision, or • in the proximity of small children.

The MEGAMAT MCZ drive can be used by children of 8 years and older, persons with reduced physical, sensory or mental capabilities, or persons with lack of experience or knowledge when they are supervised or instructed concerning the safe use of the device and when they understand the resulting risks. Do not allow children to play with this device. The cleaning and user maintenance must not be carried out by children without supervision.

	 CAUTION
	You should only use spare parts which have been manufactured or approved by DewertOkin. Only these parts will guarantee a sufficient level of safety.

Optional: Mechanical reset function: lifter release (LR) or clevis quick release (GQR)

	 CAUTION
	The mechanical reset function is not a safety system and does not avert danger.

DewertOkin does not guarantee that the drive will function in the event of a power outage. It is always possible to lower the unit using either the lifter release (LR) or the clevis quick release (GQR).

If the end-product manufacturer chooses to guarantee the functionality of the end product during a power outage, then the end-product manufacturer is responsible for arranging a mechanism to ensure this functionality.

2.2 Safety notices within the installation instruction and the operating instructions for the entire machine

The manufacturer of the end product is only permitted to operate the MEGAMAT MFZ drive (by itself an incomplete machine)

- when the end product (for which the MEGAMAT MFZ drive is intended) is in compliance with all protective measures specified in the Machinery Directive 2006/42/EC, and
- when the manufacturer expressly declares the compliance of the end product.

The manufacturer of the end product must create a manual for the users of that product. The safety notices in the end-product manual must be written based on the end product's risk assessment.

2.3 Selection and qualification of personnel

This drive should only be installed into the end product by someone who has completed training in electronic motor assembly or has equivalent qualifications.

You should only install this drive when you are qualified to do so. Otherwise, a properly qualified person should be found for this task.

2.4 Notice on safety during operations

Basic safety rules must be followed in order to ensure that the end product can be continually operated in a safe manner. These rules must be observed while using the end product and while installing the drive.

These rules and safety measures can be categorized as follows:

- Construction measures before the installation (refer to the "Ensuring operational reliability during installation" section in Chapter "Installation").
- Safety fundamentals during the drive installation and during cable and wire routing (refer to the "Safety notices to observe during installation" section in the "Installation" Chapter).
- Using the drive in intermittent duty (refer to the "General information" section in the "Operating notes" chapter).
- Basic safety rules during operation (refer to the "Operating notes" Chapter).
- The creation of a manual for the end product which contains these and other safety rules.

Creating a user's manual

The manufacturer of the end product must create a manual for the users of that product. The safety notices in the end-product manual must be written based on the end product's risk assessment.

2.5 Product identification

2.5.1 Ratings plate

A ratings plate on each drive specifies the exact name and serial number of the drive. It also states the technical specifications valid for that particular drive. The following illustration shows where the specifications are located on the drive's ratings plate.

- The ratings plate shown is an example; the specifications for your drive may differ from this illustration.

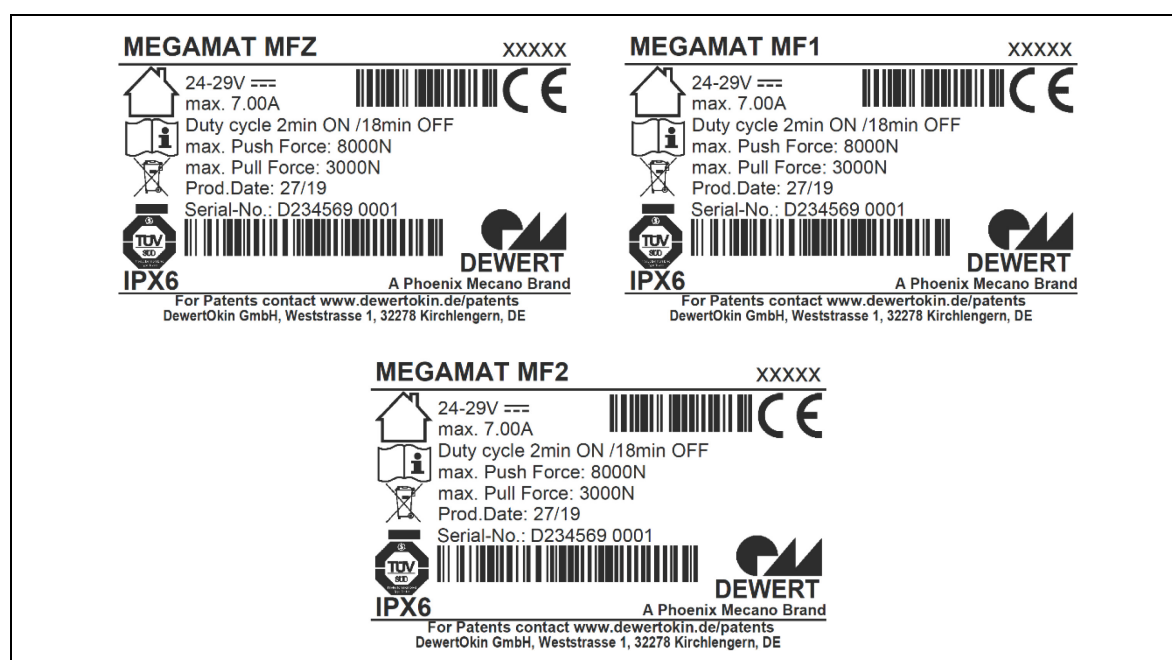





Figure 2 Ratings plate example

MEGAMAT MFZ	Model name
MEGAMAT MF1	
MEGAMAT MF2	
xxxxxx	Article number
24 – 29V ~	Input voltage
Max. 7.00A	Current consumption
Duty cycle: 2 min ON / 18 min OFF	Intermittent operations: 2 minutes / 18 minutes
max. Push Force	Push force
max. Pull Force	Pull force
Prod.date	Calendar week / year
Serial No.	Serial number for your drive
IPX6	Protection degree
	Use in dry rooms only!
	Follow all special disposal instructions!
	Conformity mark

2.5.2 Safety symbol

The symbol below is attached to the drive and has the following meaning:



This symbol indicates that the unit should only be opened by qualified specialists!

3. Possible combinations

The MEGAMAT MFZ single drive can be combined for use with other single drives, double drives or control units. The following basic combinations are possible:

- a MEGAMAT MFZ with handset and control unit,
- a MEGAMAT MFZ in combination with other drives with an additional control unit or double drive.

Systems can be customized by combining drives with the handset and control units as needed. Make sure that you switch off the electricity before connecting any components (connect while in a voltage-free state).

DewertOkin has separate system instruction manuals containing all information and instructions needed for these systems.

- ▶ Only a DewertOkin device should be used to control the drive since they have already been verified to work together.
- ▶ A control unit or a double drive is needed to operate the MEGAMAT MFZ single drive. A handset is also required.



NOTICE

Check with DewertOkin for approval concerning any special device configurations that are customized for your requirements.

4. Description

The MEGAMAT MFZ drive is an electrically driven motor that is responsible for moving the end product in a linear direction. This adjustment is created by the in and out movement of a stroke pipe. The movements of the MEGAMAT MFZ drive are controlled using a control unit or a double drive which is connected to a DewertOkin handset.

The different drive models vary according to their:

- the clevis,
- power of the drive,
- mounting of the stroke pipe,
- electrical connection.

► We reserve the right to make unannounced technical changes in the course of our continual product improvement process!

4.1 Components

The main components of the MEGAMAT MFZ drive are the motor and the stroke pipe. The stroke pipe is protected by the drive housing when it is retracted. Mounting fixtures are located at the outer end of the stroke pipe and at the opposite end of the drive. They are used to mount the drive in the end product.

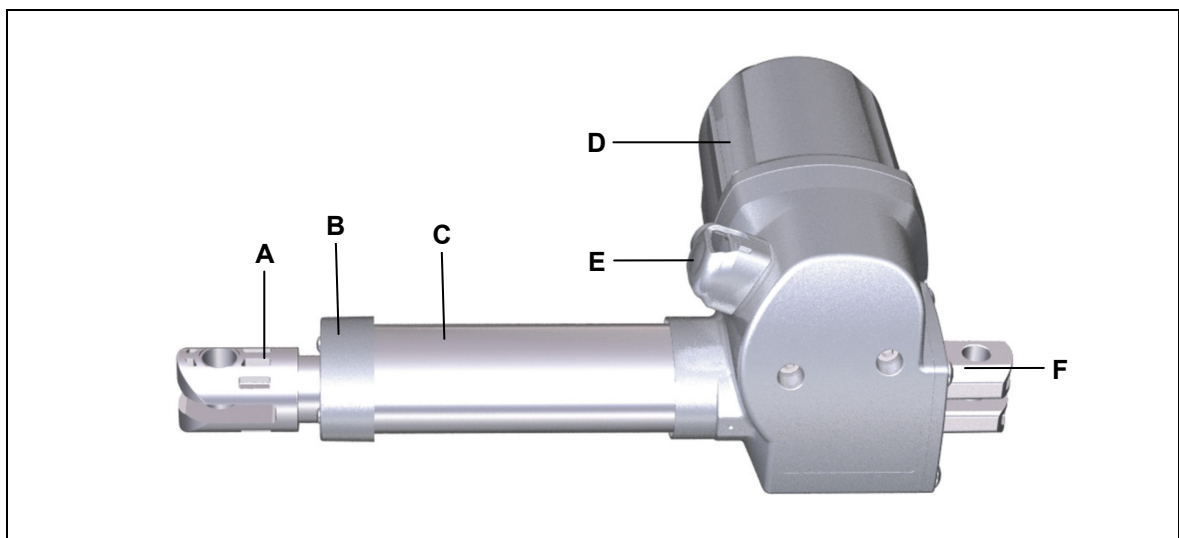


Figure 3 Main components of the MEGAMAT MFZ drive

A Clevis (example: stroke pipe clevis)

B Stroke pipe

C Guidance profile



D Motor housing

E Electrical connection

F Gear clevis

4.1.1 Stroke pipe options

There are two types of stroke pipes available for different types of applications. The stroke pipe can be screwed onto the spindle screw or it can be mounted unattached. The drive only exerts pushing force when the stroke pipe is mounted loosely (unattached). The motions of the drives for these two versions are shown in the following illustrations.

	<div data-bbox="395 584 592 636"> CAUTION</div> <p data-bbox="395 656 1414 784">If the stroke pipe is unattached, then the end product must be designed to prevent the stroke pipe from coming out during operations. This can be accomplished by implementing a fixed end-stop mechanism (refer to the "Safety notices to observe during installation" in the "Installation" chapter).</p>
---	---

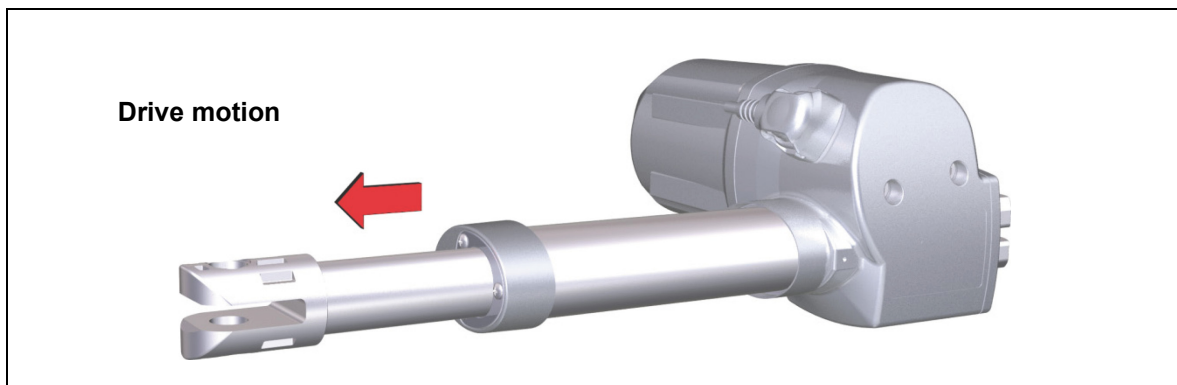


Figure 4 Loose fastening nut (stroke pipe not secured): only pushing forces

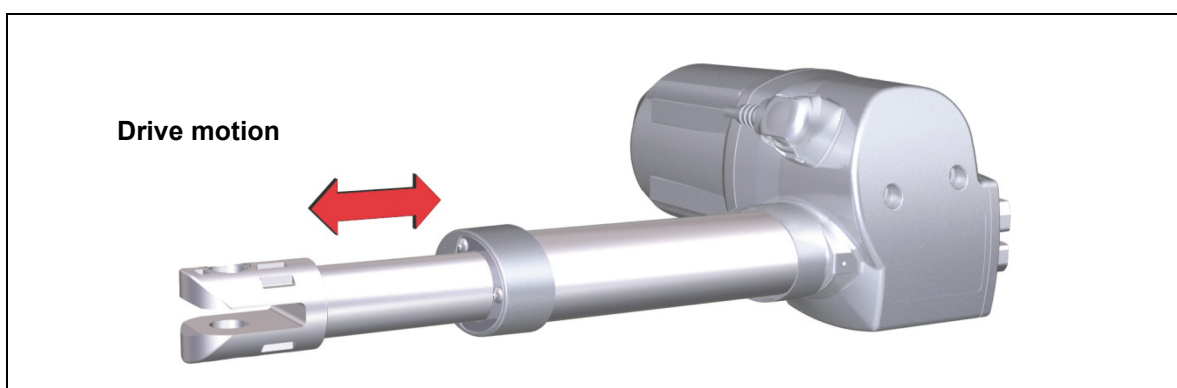


Figure 5 Stroke pipe screwed onto the spindle screw (attached): both pushing and pulling forces

4.1.2 Clevis options
Clevis versions: Stroke pipe clevis

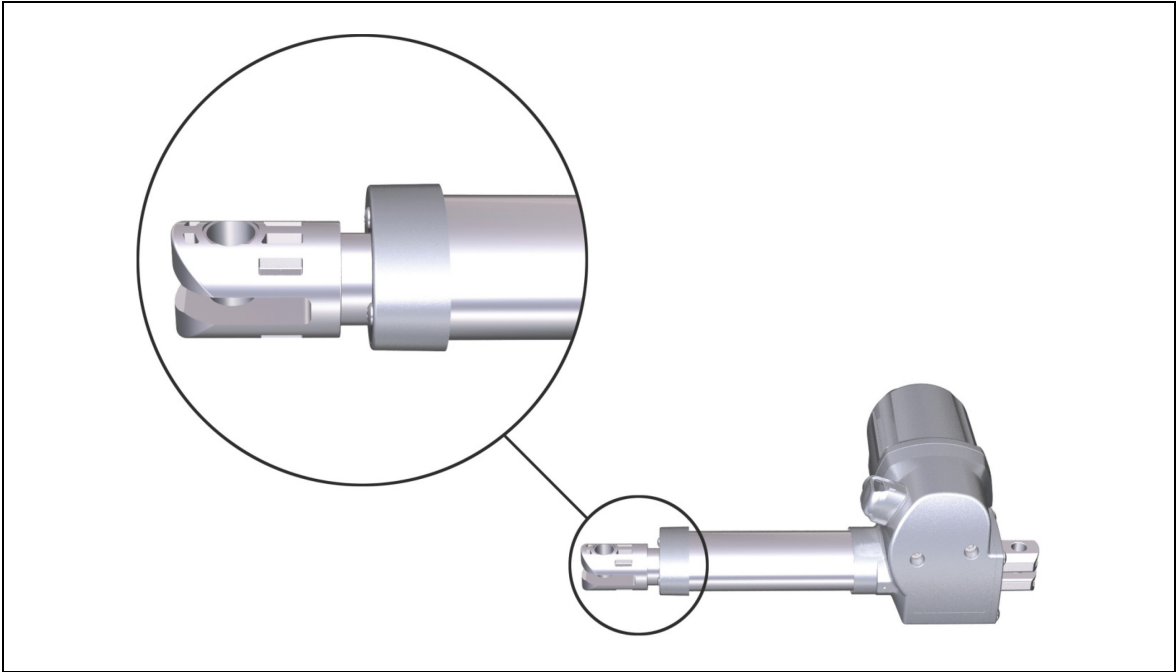




Figure 6 Clevis version with stroke pipe clevis

Clevis versions: Clevis quick release (GQR)

The clevis quick release (GQR) is a mechanical reset function that is used to manually lower the patient lifter (during a power or voltage outage, for example).

	 CAUTION
	The mechanical reset function is not a safety system and does not avert danger.

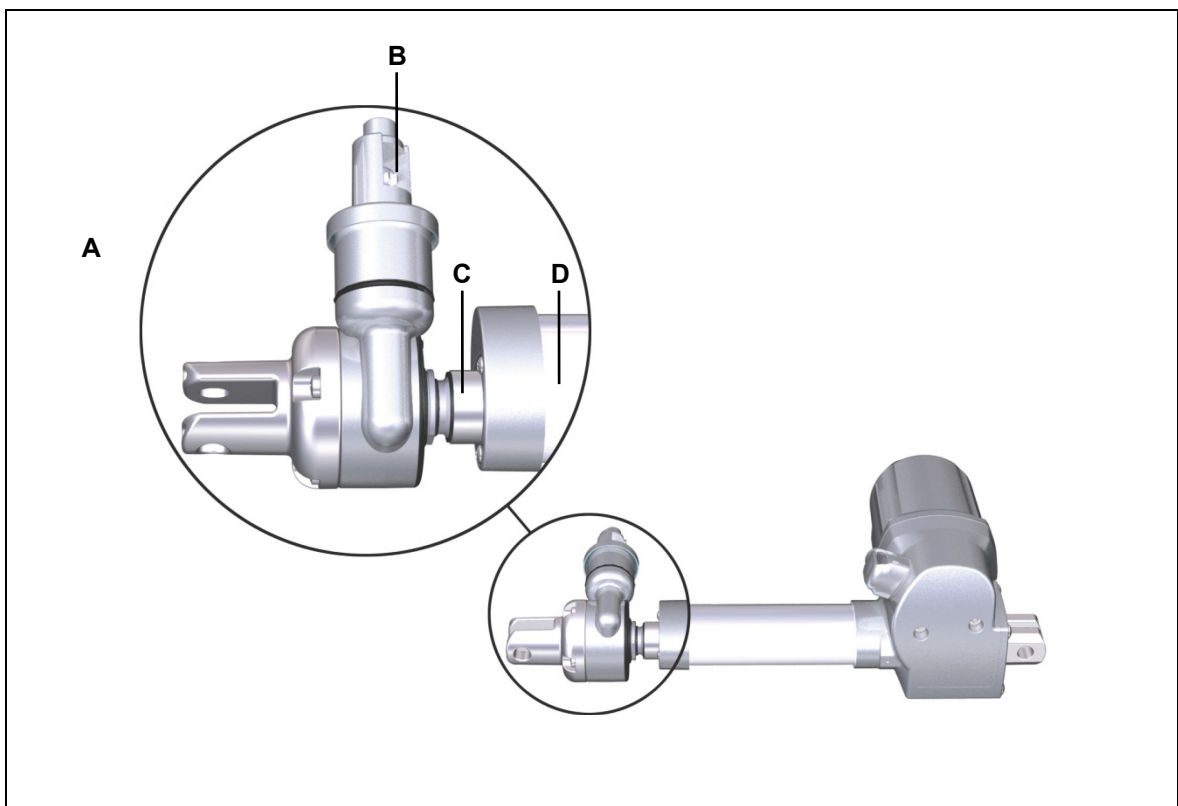


Figure 7 The GQR clevis model

A Clevis quick release (GQR)

C Stroke pipe

B Bowden cable holder

D Guidance profile

Clevis versions: Lifter release (LR)

The lifter release (LR) is a mechanical reset function that is used to manually lower the patient lifter (during a power or voltage outage, for example).

**CAUTION**

The mechanical reset function is not a safety system and does not avert danger.

**NOTICE**

The MEGAMAT MFZ drive with the mechanical reset function lifter release (LR) has a limited five-year lifespan. The actual duration of use can be determined from the date of production as specified on the ratings plate (refer to section 2.5.1).

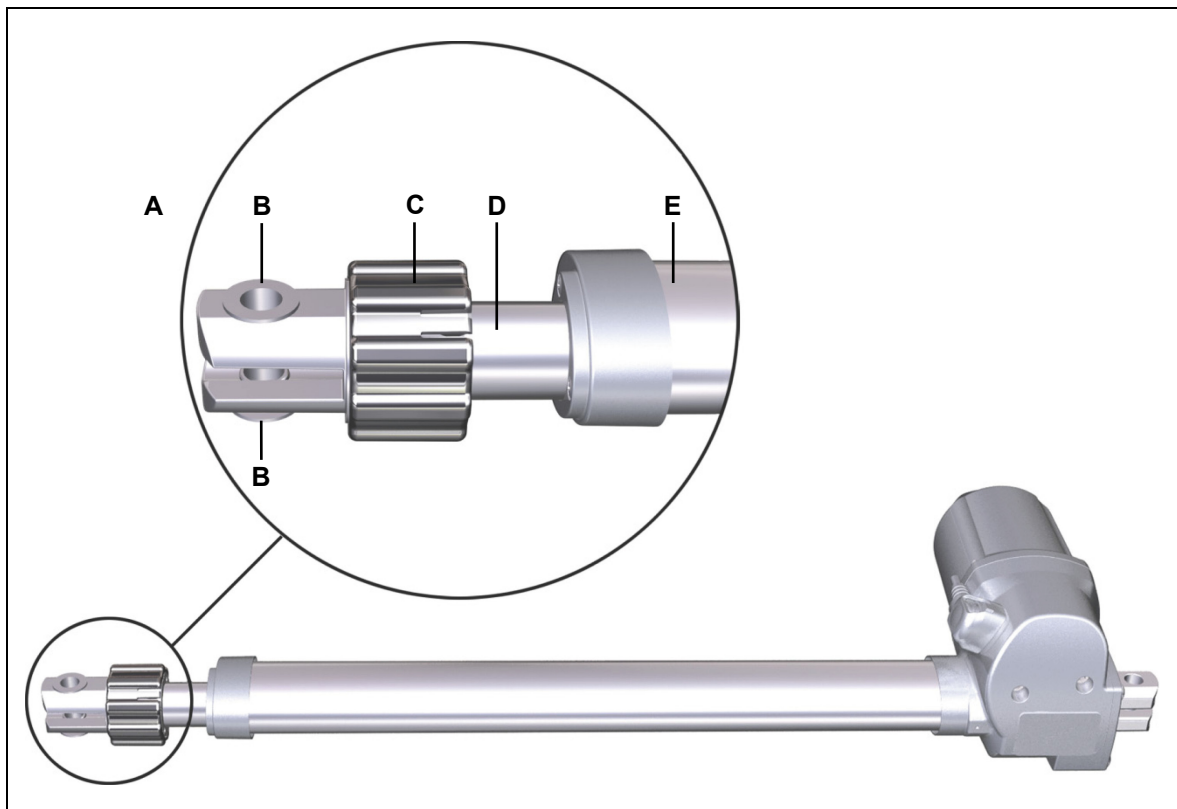


Figure 8 Clevis version, lifter release (LR)

A Clevis (lifter release)

B Bearing bushing

C Safety clutch on the lifter release

D Stroke pipe

E Guidance profile

**NOTICE**

Both bearing bushings must always be installed in the LR clevis.

4.1.3 Electrical connection options

There are two different electrical connection options available: Connections with a plug-in motor cable (detachable snap-on cap) or a fixed motor cable (non-detachable cap). For the non-detachable cap, the electrical connection is permanently attached at the factory and cannot be opened. With the detachable snap-on cap, the electrical connection may be opened by a qualified specialist. In this case, the snap-on cap has openings on the side, through which a screwdriver can be inserted to release the snap connection.

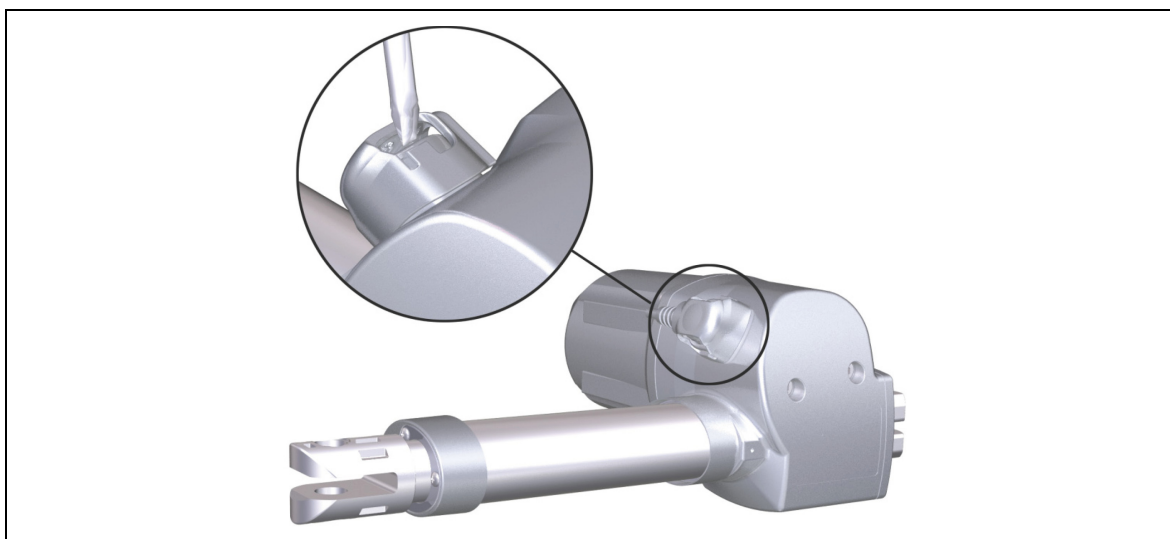


Figure 9 Detachable snap-on cap for plug-in motor cable

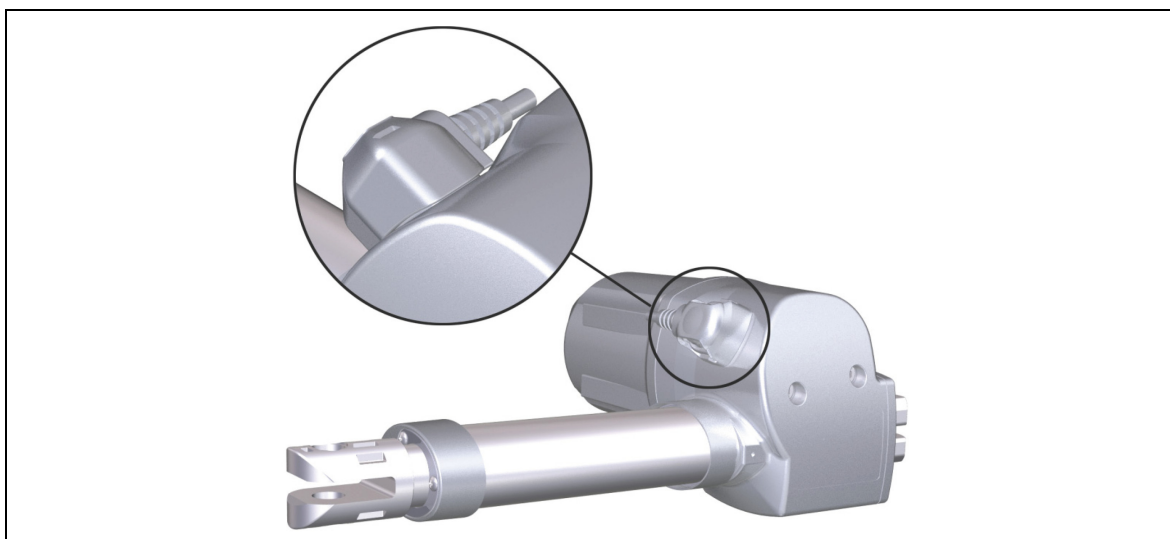


Figure 10 Fixed cap for fixed motor cable

5. Technical specifications

Rated voltage	24 V DC – 29 V DC
Current consumption at rated load	Max. 7 A, depending on version (refer to the ratings plate on the drive)
Permitted push force	Max. 8000 N, depending on version (refer to the ratings plate on the drive)
Permitted push force for GQR	Max. 3500 N, depending on version (refer to the ratings plate on the drive)
Permitted pull force	Max. 3000 N, depending on version (refer to the ratings plate on the drive)
Release force (for version with clevis quick release, GQR)	Approx. 34 kg
Rated torque for clevis lifter release	Approx. 8 Nm
Starting torque for clevis lifter release	Approx. 24 Nm
Mode of operation ¹ at max. rated load	Intermittent duty: 2 min/18 min (max 5 switching cycles per minute)
Protection class ²⁾	III
Noise level	≤ 65 dB(A)
Drive type	Single drive
Load type	Push / pull
Type of load (version with lifter release, LR)	Pressure
Stroke	< 300 mm
Adjustment speed ³	Up to 8 mm/second, depending on the version
Protection degree	IP20, IPX4, IPX6
Dimensions and weight	
Length x width x height	Min. 191 mm x 176 mm x 98 mm
Length x width x height (for version with lifter release)	Min. 261 mm x 176 mm x 98 mm
Length x width x height (for version with clevis quick release, GQR)	Min. 232 mm x 176 mm x 98 mm
Weight	Approx. 1.7 kg, depending on version

¹⁾ Mode of operation: intermittent duty 2 min./18 min. This means that after the unit is operated with its rated load for up to two minutes it must then be paused for 18 minutes. The system can malfunction if this pause is not observed!

²⁾ Safety extra-low voltage

³⁾ Adjustment speed: the speed at which the clevis can move under no load (the speed varies depending on the load).

Ambient conditions for operation, storage and transport	
Transport / storage temperature	From -20 °C to +50 °C From -4 °F to +122 °F
Operating temperature	From +10 °C to +40 °C From +50 °F to +104 °F
Relative humidity	From 30% to 75%
Air pressure	From 800 hPa to 1060 hPa
Height	< 2000 m

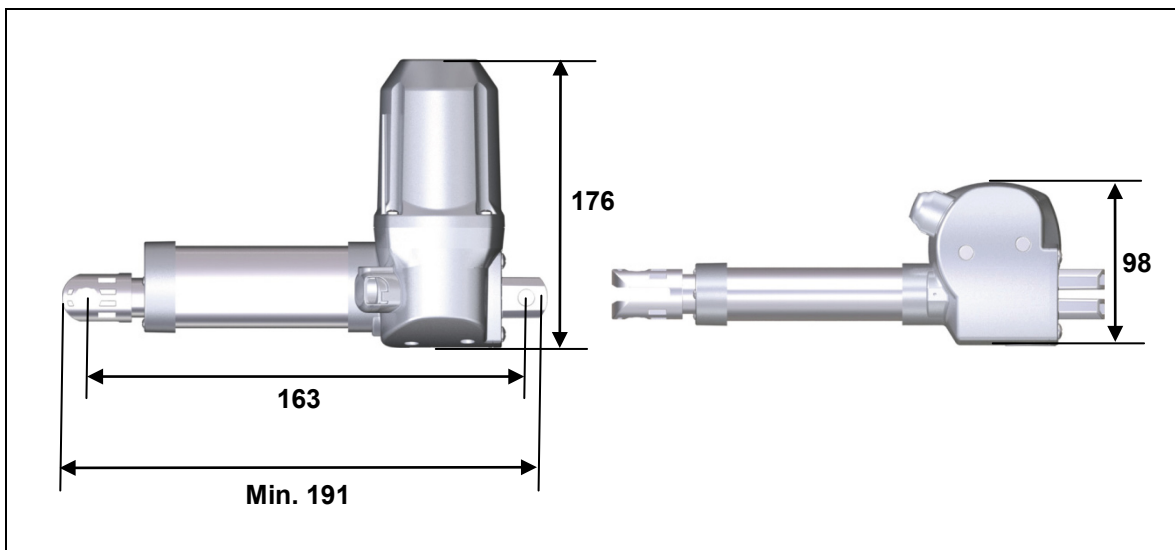


Figure 11 Dimensions of MEGAMAT MFZ drive (in mm),
minimum installation dimension: 163 mm

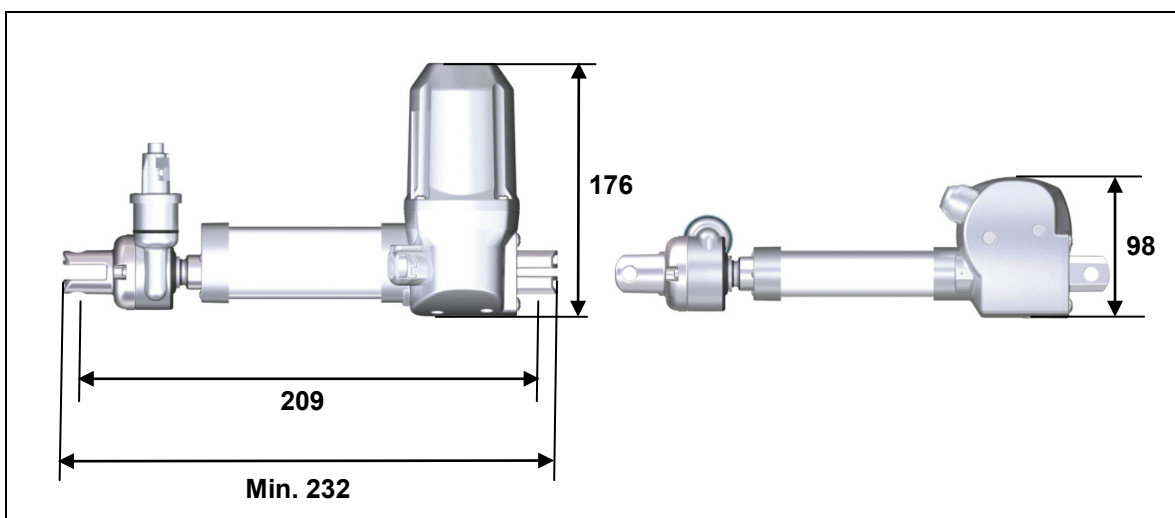


Figure 12 Dimensions of MEGAMAT MFZ drive with GQR (in mm),
minimum installation dimensions: 209 mm

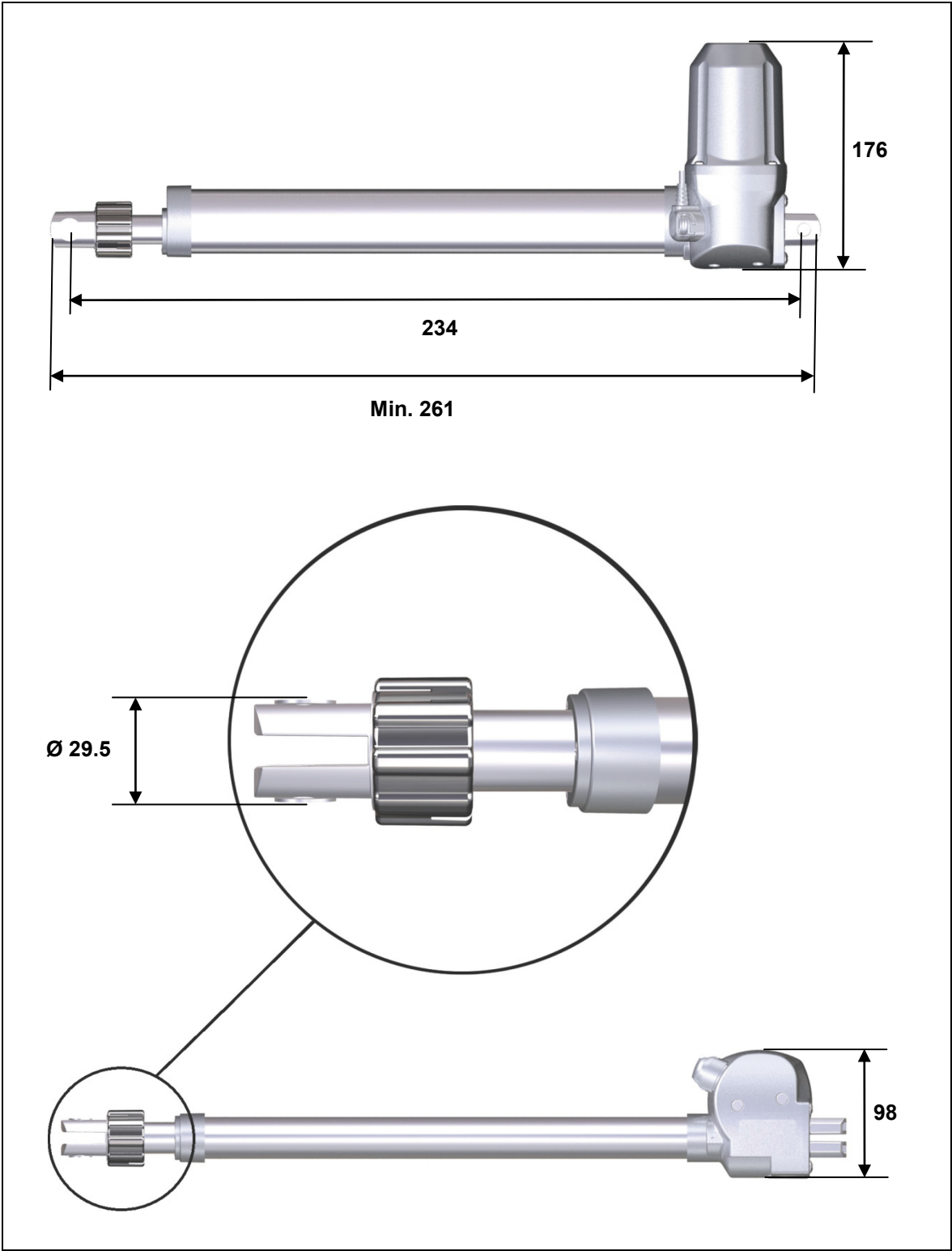


Figure 13 Dimensions of MEGAMAT MFZ drive with LR (in mm), minimum installation dimension: 234 mm

6. Installation



6.1 Safety notices to observe during installation

Basic safety rules must be followed in order to ensure that the end product can be continually operated in a safe manner. These rules must be observed while using the end product and while installing the drive.



6.1.1 Ensuring operational reliability during installation

The safety and reliability of the end product containing the DewertOkin drive can be ensured by using the proper construction methods described below.

Avoiding fatigue fractures



	 CAUTION
	<p>Drives that are incorrectly installed can undergo fatigue fractures which then create a risk of injury.</p> <ul style="list-style-type: none"> • Install the drive in the end product so that it is properly aligned. This will help prevent shear stress. • Do not position the drive at a slanted angle when installing it in the end product. A slanted angle between the intended direction of movement of the end product and the drive's direction will create shear stress and could lead to a fatigue fracture. • Be sure to install the drive so that it can always move freely in all operative states. • You should never screw the stroke pipe into the drive (versions with fixed nuts).

Avoiding a pinching hazard

	 CAUTION
	<p>A stroke pipe is used to provide the drive motion. When designing your product, you should take this into account with passive safety mechanisms and with the appropriate safety notices in your operating instructions:</p> <ul style="list-style-type: none"> • Installation methods for ensuring passive safety: Install the MEGAMAT MFZ drive so that none of the positions where shear and pinch hazards exist are accessible externally.

When preparing safety notices for the operator, Make sure that your operating instructions inform the user of these safety points.

Avoiding an over-extension of the stroke pipe (when mounted unattached)


	 CAUTION
	<p>There must be a fixed mechanical end stop built into the end product for MEGAMAT MFZ drive models where the stroke pipe is not attached to the spindle screw. If no fixed mechanical end stop exists, the stroke pipe can over-extend. This can cause malfunctions, personal injury, and damage to the drive system.</p>

Avoiding an overrun of the stop point with the end product

Your end product should contain mechanical end stops. These will limit the drive movement and significantly increase operational safety. DewertOkin recommends that you build such mechanical end stops into your end product.

If, for technical or cost reasons, you choose not to use end stops, then we strongly recommend the use of an additional safety limit switch whenever the DewertOkin drive is being used for at-risk functions. A safety limit switch will protect the drive from a critical failure and overrun in the event that the top end switch malfunctions.

- ▶ The safety end switch shuts the drive down as soon as the normal end switch is passed over. The safety end switch is integrated directly into the drive and does not require any additional installation. The drive may take up an extra 23 mm when installed with the safety limit switch.

	NOTICE
	<p>When installing the optional clevis quick release (GQR), you must use mechanical end stops (both for the upper limit and for the lower limit) in your end product in order to prevent mechanical damage to the drive system.</p>

- ▶ The mechanical end stops or stroke limiters in the end product must be adjusted to fit the retracted and extended dimensions of the drive. Make sure that the drive does not reach the mechanical end stop position before it has been fully extended.

6.2 Installation procedure

NOTICE

Mount the drive in its unloaded position within your end product.
The drive must either be completely retracted or extended at its end position.
Damage to the drive can **only** be avoided if the drive is installed in its unloaded position. This ensures safe installation.

6.2.1 An installation example

Before installing the drive, make sure that you are observing all of the safety notices found in the "Safety notices to observe during installation" section.



CAUTION

The plug for the connecting cable should be unplugged during the installation.

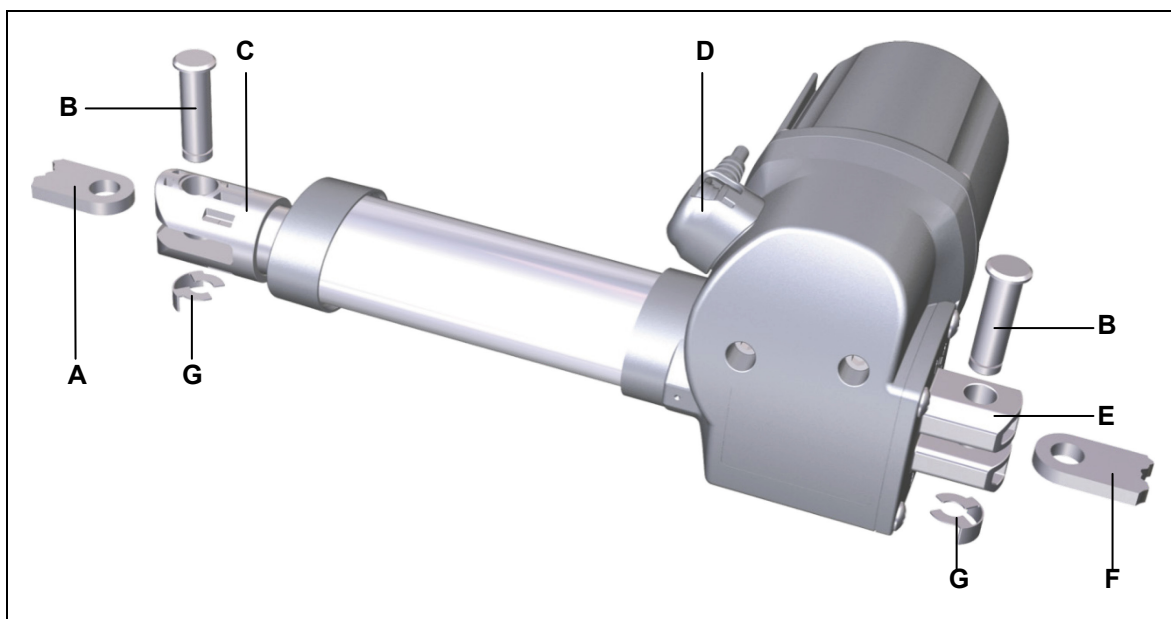


Figure 14 Installing the drive (example)

A Bracket for securing the drive (on the stroke pipe side)

C Stroke pipe clevis

E Gear clevis

G Safety clip

B Mounting bolt

D Electrical connection / cable

F Bracket for securing the drive (on the motor side)

**NOTICE**

The support brackets for the clevises must be in alignment.

In the following example, installation of the MEGAMAT MFZ drive into the end product is shown using mounting bolts.

► Certain details may change based on the variation in use and technical modifications.

- 1 Move your product into a position where it is supporting no load.
- 2 Push the drive's gear clevis (E) onto the bracket (F).
- 3 Insert the mounting bolt (B) into the bracket (F).
- 4 Secure the mounting bolt (B) with the safety clip (G).

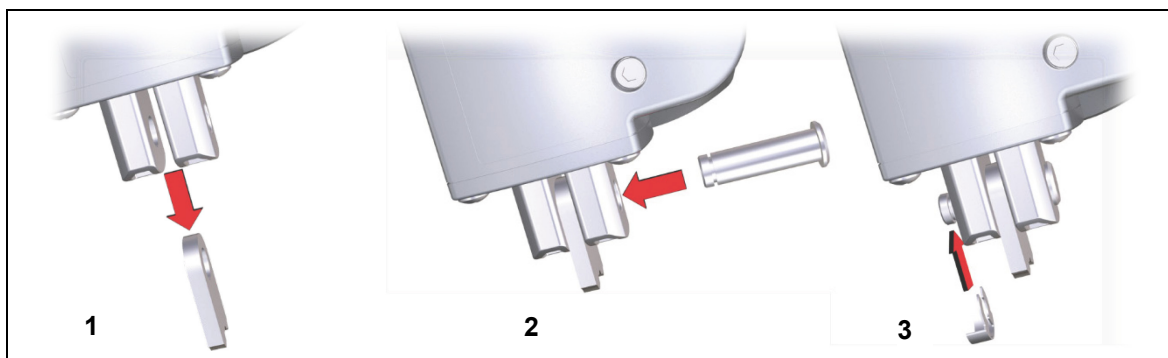


Figure 15 Steps for installing the gear clevis (example)

**CAUTION**

Versions with fixed nuts: You should never screw the stroke pipe into the drive. A stroke pipe that has been screwed into the drive can result in fatigue fractures.

- 5 if the stroke pipe clevis/head (C) does not align exactly with the gear clevis (E), the stroke pipe may be unscrewed out of the drive by no more than a half turn.

**CAUTION**

Versions with fixed nuts: The stroke pipe may only be turned one half rotation out of the drive. Do not rotate the stroke pipe further out. An over-rotation can lead to unsafe operations. The stroke pipe clevis must not be detached from the stroke pipe!

- 6 Attach the stroke pipe clevis (C) using a mounting bolt (B) to the mounting bracket (A).
- 7 Secure the mounting bolt (B) with the safety clip (G).
- 8 Now connect the drive to the control unit or the double drive. Ensure that the electrical connection cable (D) cannot be pulled out.

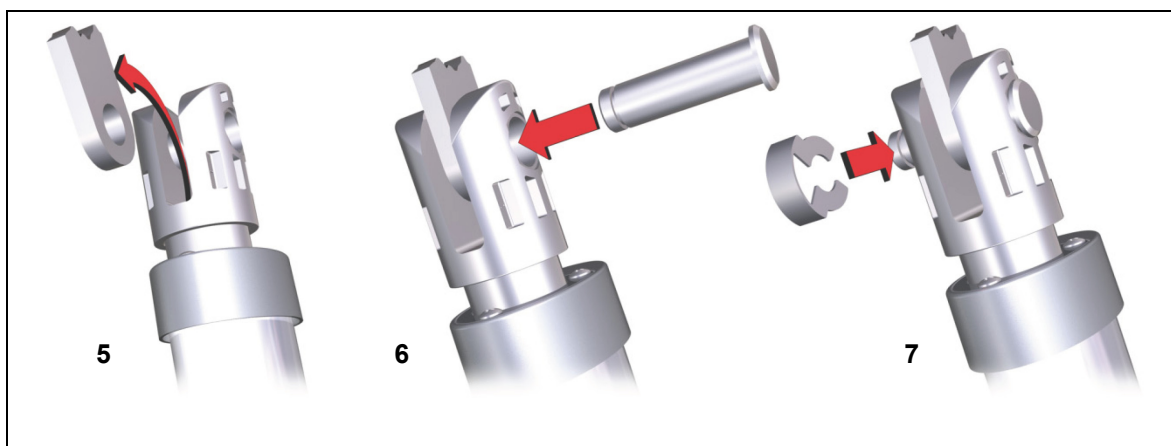




Figure 16 Steps for installing the stroke pipe clevis (example)

- If the hand switch will be used to control two drives, you must first connect both drives electrically to each other (for example, over a control unit or double drive). Then you can connect the handset.

6.2.2 Information on installing the MEGAMAT MFZ drive with clevis quick release (GQR)

	 CAUTION
	<p>Installation errors (due to deviations in the brackets or fittings, for example) can significantly alter the release forces of the GQR. Improper installation can lead to significantly increased release forces which may far exceed the specifications.</p>

Position and alignment of the bracket on the clevis quick release (GQR)

The support brackets for the clevis must be properly aligned (refer to Figure 17). If they do not line up, additional lateral forces will be exerted on the drive unit. If the brackets are out of alignment by only one degree, the release forces can increase significantly. Ideally the brackets should be fitted at almost zero degrees alignment.

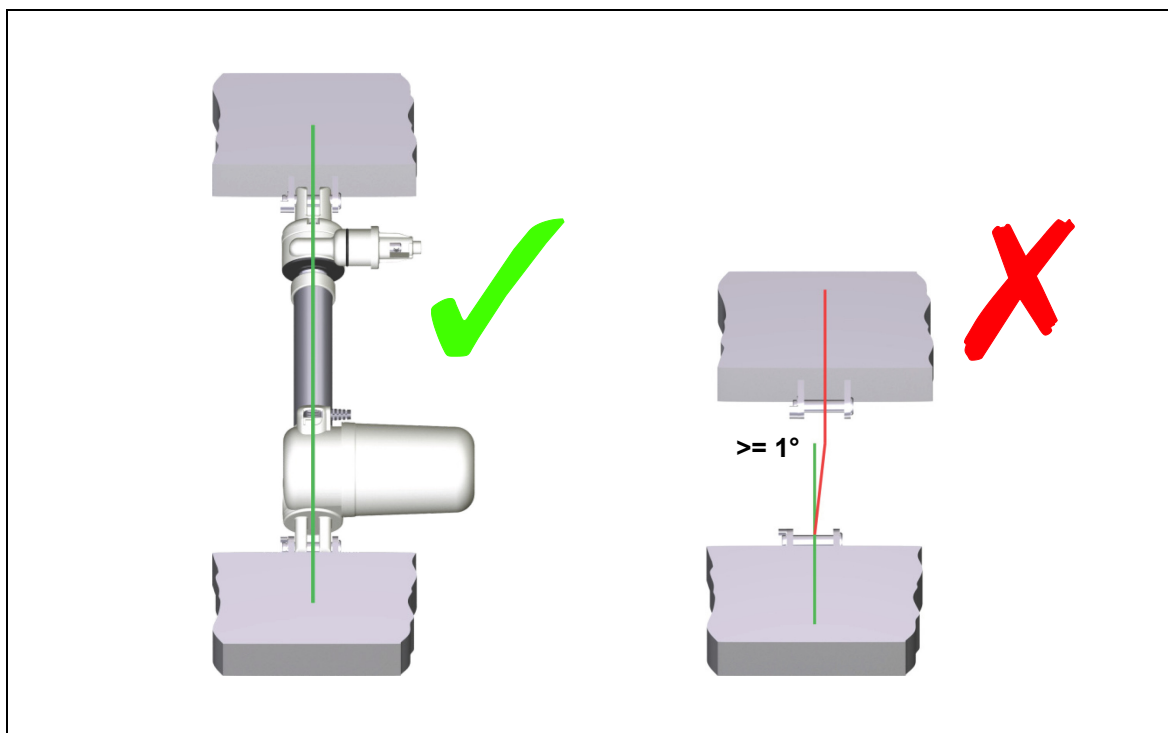


Figure 17 Position and alignment of the brackets

Installation position of the clevis quick release (GQR)

- The release force will increase, for example, if the drive unit is subjected to a lateral force from other attachments. The resulting release force may be far over the specification. (Figure 18 shows a horizontally installed drive unit with a lateral force acting upon it.)

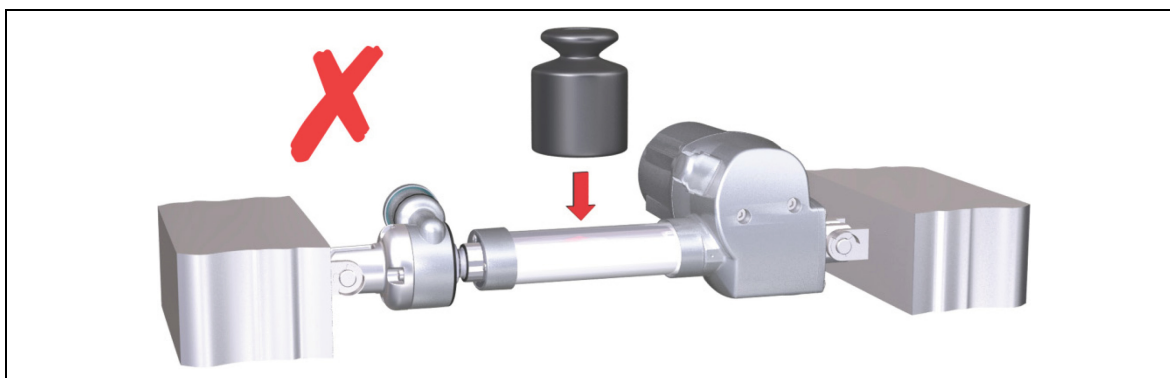


Figure 18 The effect of the installation position

Recommended bracket position for the clevis quick release (GQR)

The brackets should be aligned precisely with each other. We recommend that only one mounting bracket is used on both sides of the drive (as shown by **A** in Figure 19). When two mounting brackets are used, there is a greater risk that they are not properly aligned, resulting in a tilted drive unit.

The drive must be firmly attached, but there should also be a certain degree of movability at the mounting points. The drive must be mounted so that it is protected in all positions against twisting and bending. You can also use brackets with two mounting holes (as shown by **B** in Figure 19), but these must be exactly aligned so that the release force is not increased.

The mounting bracket should have an elongated oval hole so that you can compensate for any deviations in the installation dimensions of the drive and your end product.

- We recommend: an oval slot (as shown by **C** in Figure 19) should have at least 2 millimetres of extra room.

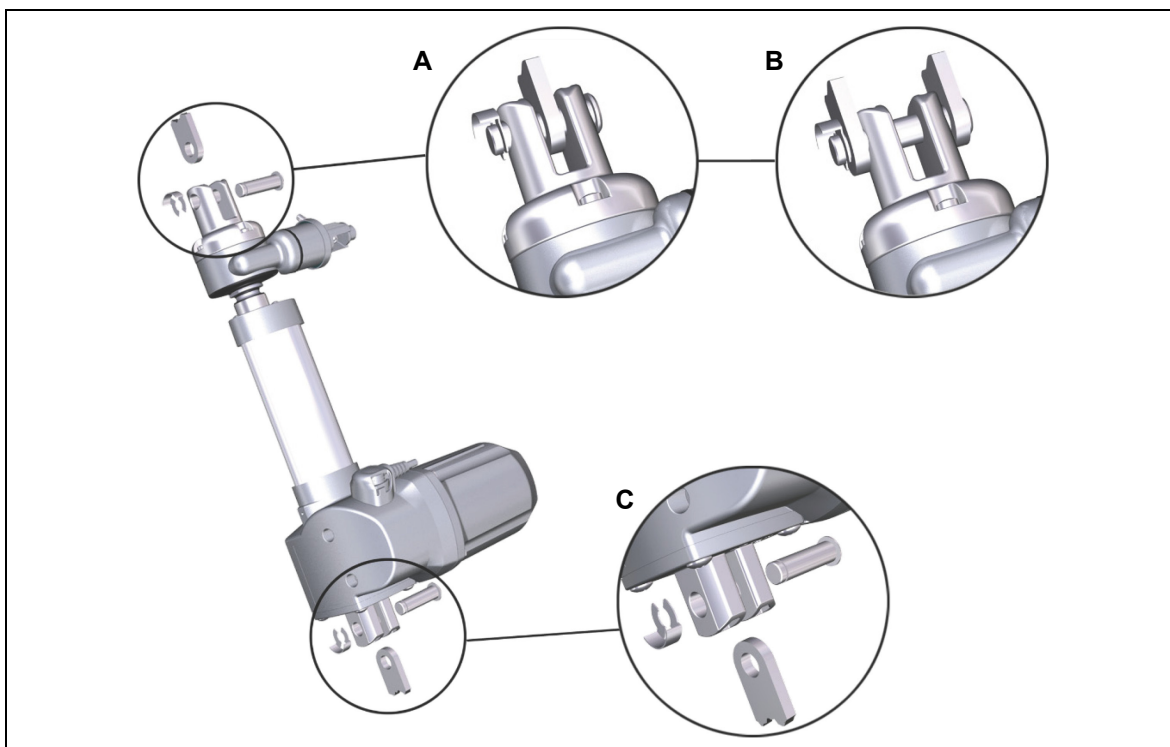


Figure 19 Recommended design of bracket

6.2.3 Installation of the Bowden cable for the optional mechanical reset function GQR



NOTICE

When installing the GQR, use a Bowden cable with some type of length compensation mechanism (for example, attach a spring to the Bowden cable).

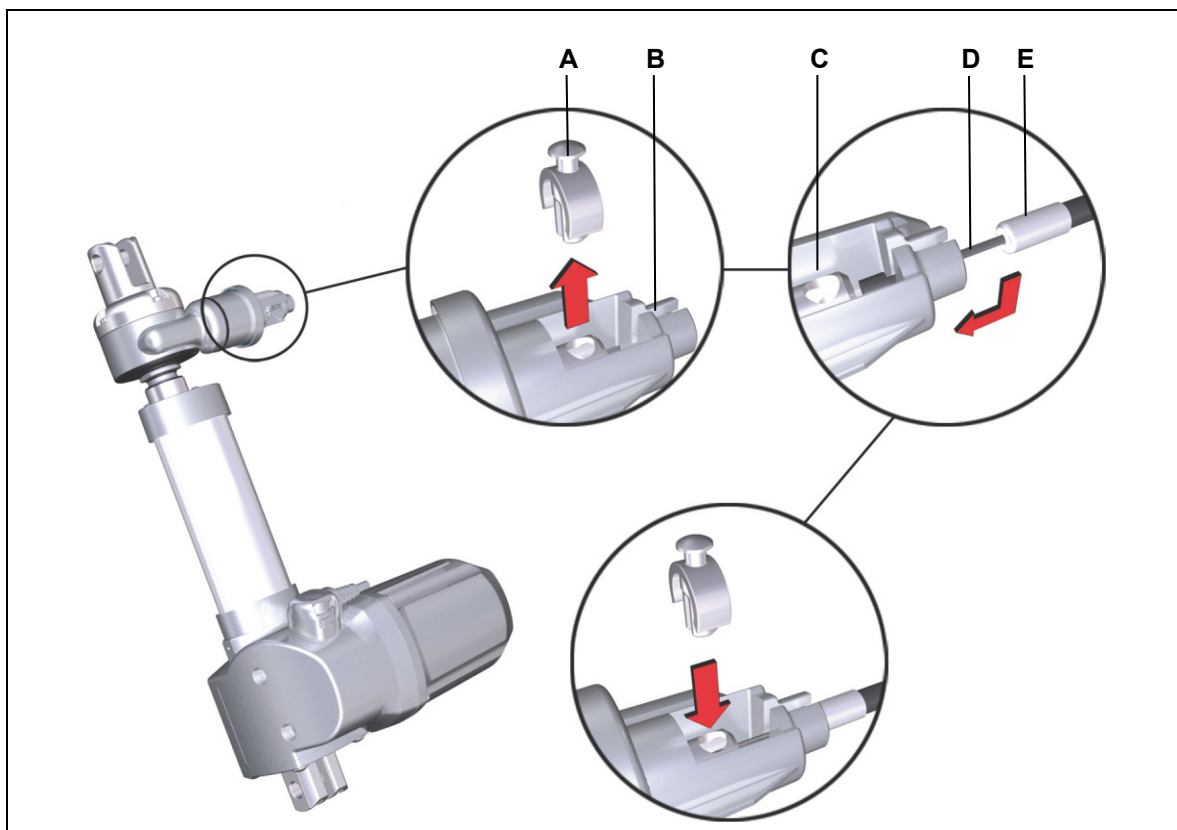


Figure 20 Installation of the Bowden cable for the mechanical reset function GQR (optional)

A Bowden cable lock

B Groove in the GQR

C Bowden cable holder

D Bowden cable with length compensation mechanism (e.g. a spring)

E Bowden cable sheath

- 1 Guide the Bowden cable (**D**) through the slot in the GQR (**B**).
- 1 Place the Bowden cable (**D**) into the cable holder (**C**).
- 2 Secure the cable in position using the locking mechanism (**A**).





CAUTION

Do not put too much tension on the Bowden cable. An over-strained Bowden cable could cause an accidental adjustment to occur by triggering the mechanical reset function.

- ▶ The Bowden cable holder (C) must be pulled until it reaches the end stop (all the way up) in order to ensure that the GQR functions properly. The GQR's clutch opens up when the pull force reaches 110 N. The pull force must not exceed 250 N, otherwise the GQR could be damaged.
- 3 Mount the Bowden cable to your end product. A spring should be attached to the Bowden cable mechanism to prevent damage to the GQR and to ensure proper function.
- 4 Verify that the mechanical reset function is functioning properly.

6.2.4 Information for installing the MEGAMAT MFZ drive with lifter release (LR) in a safe position

	 CAUTION
	Installation errors (e.g. due to deviations in the brackets or fittings) can significantly alter the release forces of the lifter release (LR) mechanism. Improper installation can lead to significantly increased release forces which may far exceed the specifications.

Position and alignment of the bracket on the lifter release (LR)

The support brackets for the clevis support at the lifter release (LR) must be properly aligned (refer to Figure 21). If they do not line up, additional lateral (shear) forces will be exerted on the drive unit. If the brackets are out of alignment by only one degree, the release forces can increase significantly. Ideally the brackets should be fitted at almost zero degrees alignment.

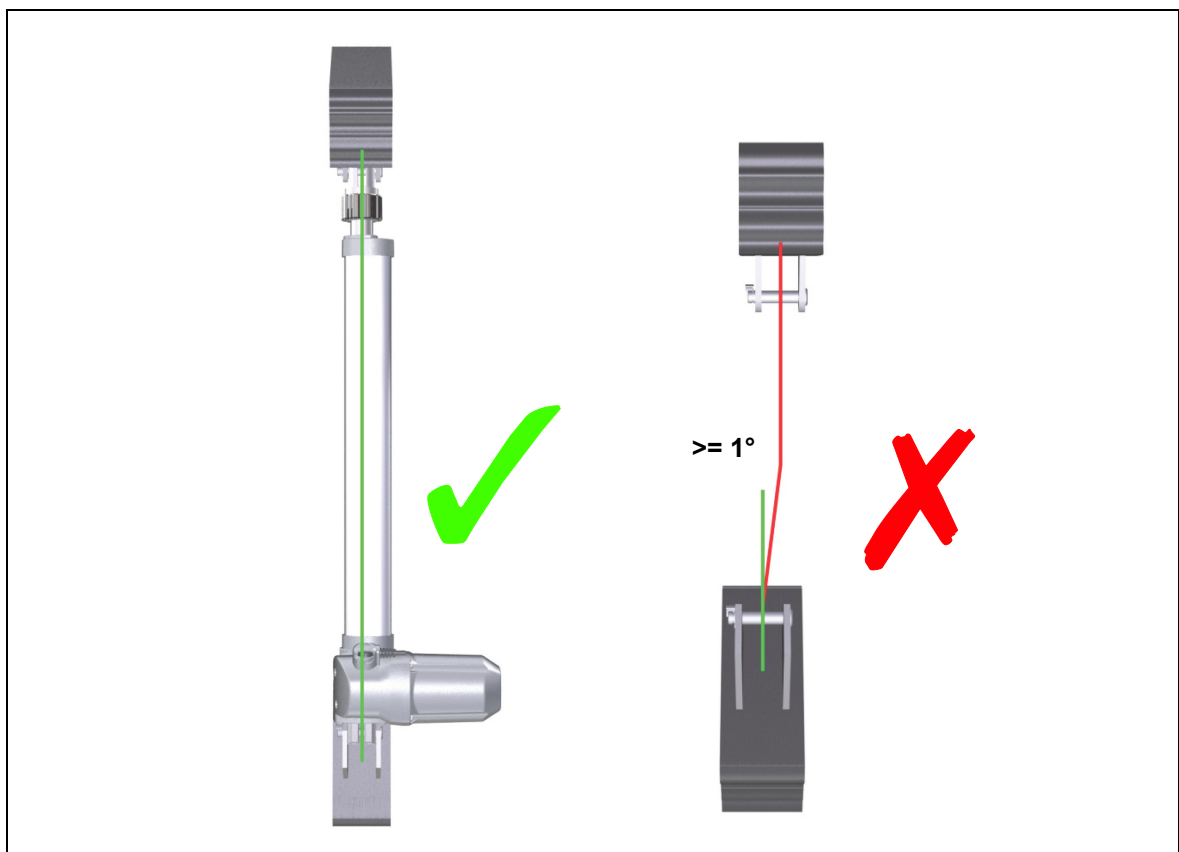


Figure 21 Position and alignment of the bracket on the lifter release (LR)

6.2.5 Installation of MEGAMAT MFZ drive with lifter release (LR) in the patient lifter (example)

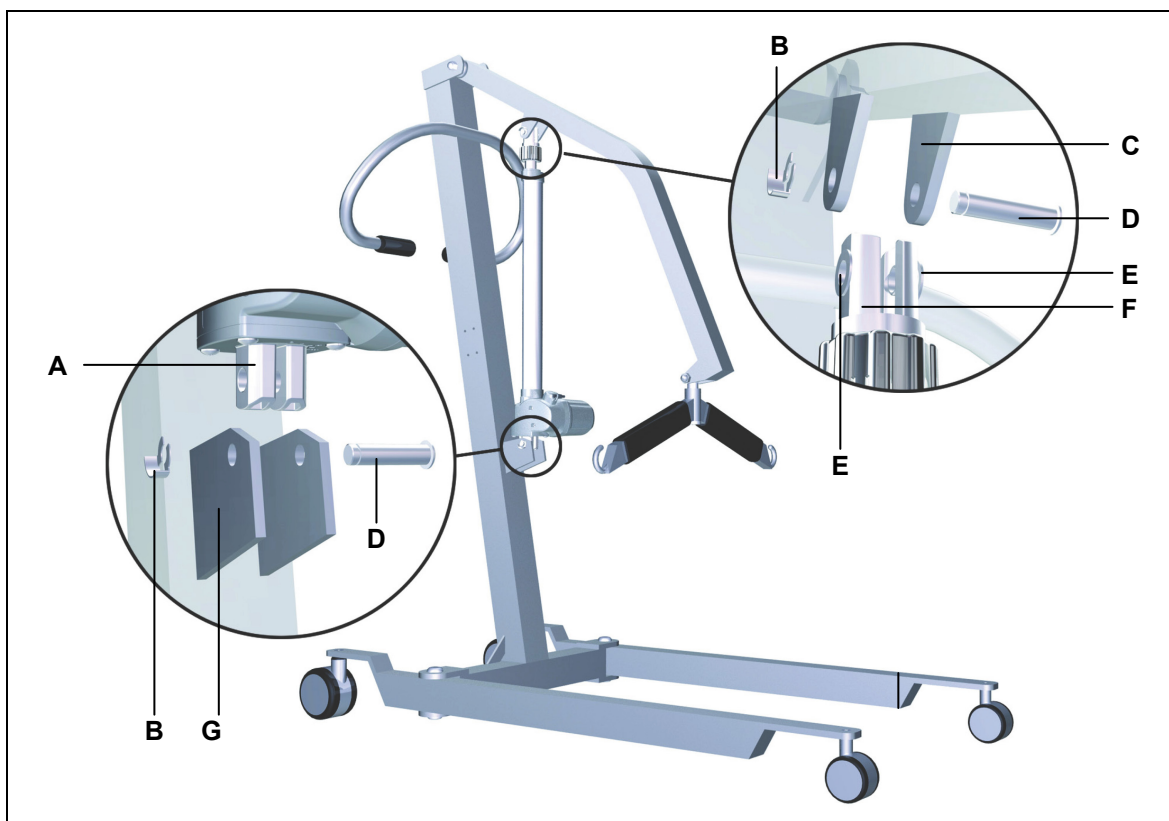


Figure 22 Overview: Installation in the patient lifter (example)

- | | |
|---|------------------------|
| A Gear clevis | B Safety clip |
| C Bracket for securing the drive (on the stroke pipe side) | D Mounting bolt |
| E Bearing bushing (two) | F Clevis (LR) |
| G Bracket for securing the drive (on the motor side) | |



CAUTION

Electrical components should be connected or disconnected only when the mains power plug and the battery plug (when present) are both unplugged.

► Certain details may change because of technical changes.

- 1 Push the drive's gear clevis (**A**) onto the bracket (**G**).
- 2 Insert the mounting bolt (**D**) into the bracket (**G**).
- 3 Secure the mounting bolt (**D**) with the safety clip (**B**).

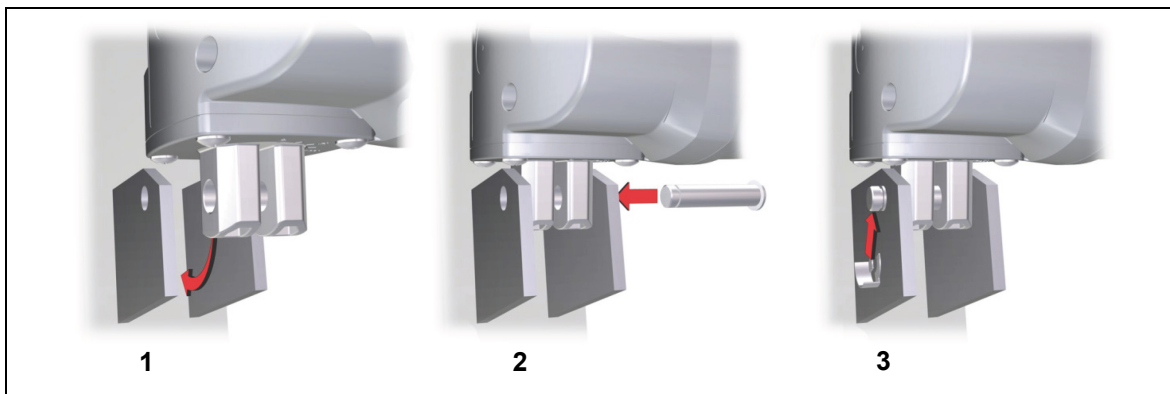


Figure 23 Steps for installing the gear clevis (example)

- 4 When the LR clevis (**F**) is not properly aligned with the gear clevis (**A**) then it must be re-aligned.



NOTICE



Both bearing bushings must always be installed in the LR clevis.

- 5 Make sure that both bearing bushings (**E**) are installed on the LR clevis (**F**).
- 6 Attach the LR clevis (**F**) using a mounting bolt (**D**) to the bracket (**B**).
- 7 Secure the mounting bolt (**D**) with the safety clip (**B**).
- 8 Now connect the drive to the control unit.



Figure 24 Steps for installing the LR clevis (example)

6.2.6 Electrical connection

	 CAUTION
Electrical components should be connected or disconnected only when the mains power plug and the battery plug (when present) are both unplugged.	



Routing the electrical cables

When routing the cables, be sure that:

- the cables cannot get jammed,
- no mechanical load (such as pulling, pushing or bending) will be put on the cables, and
- the cables cannot be damaged in any way.

Fasten all cables (especially the mains cable) to the end product using sufficient strain relief and kink prevention methods. Be sure that the design of the end product prevents the mains cable from coming into contact with the floor during transport.

6.2.7 Connection option: Drive with detachable snap-on cap covering the electrical access

	 WARNING
<p>Only personnel with the following training are qualified to work on the electrical connections:</p> <ul style="list-style-type: none">• someone who has completed training in electronic motor assembly,• someone with equivalent qualifications, or• someone who has successfully completed the appropriate DewertOkin training program. <p>You should only work on the electrical connections if you are qualified to do so. Otherwise, a properly qualified person should be found for this task.</p>	

The snap-on cap has openings on the side, through which a screwdriver can be inserted to release the snap connection.

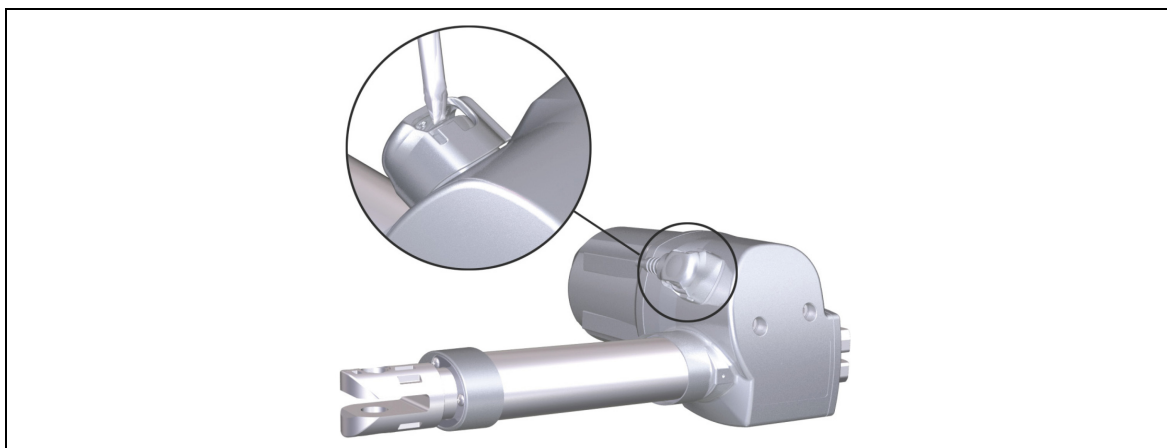




Figure 25 Detachable snap-on cap

6.2.8 Connection option: Drives with a fixed cap covering the electrical access



	 WARNING
	You may not open up the electrical access connection if the electrical access is covered with a permanent cap instead of a snap-on cap.



The following symbol can be found on the MEGAMAT MFZ drive's snap-on cap.



This symbol indicates that the unit should only be opened by qualified specialists!


6.2.9 Removing the drive

	 CAUTION
	Electrical components should be connected or disconnected only when the mains power plug and the battery plug (when present) are both unplugged.

	 CAUTION
	Be sure to carry out work on the drive in a position so that no loads are bearing on it. Only in this way can you be sure to avoid any risks of crushing or injury.

► Certain details may change because of technical changes.



- 1 Move the end product (patient lifter) into the no-load position.
- 2 Remove the connecting cable from the controller or double drive.
- 3 If multiple drives are connected electrically, they should be disconnected from each other.

	NOTICE
	Be sure to support the drive's weight while you are loosening the bolt.

- 4 Remove the security clips and the mounting bolt.
- 5 Remove the drive from the end product (patient lifter).

7. Operating notes

The factual information contained within may be used when you are creating the end-product manual. The installation instructions do not contain all information required for the safe operation of the end product. They only describe the installation and operation of the drive as a partially assembled piece of machinery.

	 CAUTION
	When creating the operating instructions, remember that the installation instructions are intended for qualified specialists and are not for typical users of the end product.

7.1 General information



- Only a DewertOkin device should be used to control the drive since they have already been verified to work together.

Power-on time / intermittent operations



The MEGAMAT MFZ drive has been designed for intermittent operations. Intermittent operation is an operational mode where the drive must pause after a specified maximum period of operation (power-on time). This protects the drive from overheating. Extreme overheating can cause a malfunction.

- The ratings plate specifies the maximum power-on time and the required pause intervals.

Avoiding electrical risks



	 WARNING
	Make sure that all live (current-carrying) parts of the drive system and power supply cannot be touched. In particular, be sure that unused power and control unit connections are covered adequately.

Shutting off the drive

	 CAUTION
	In order to shut off the drive, unplug the mains power plug and the battery plug (when present). The power plug (of the control unit or double drive) must always be accessible during operations so that emergency shut-off is possible.



Avoiding damage to the drive

Be sure that your operating instructions inform the user about the possible drive risks.



	 CAUTION
	Make sure that no lateral force is exerted on the MEGAMAT MFZ drive. Such forces can occur, for example, if the drive is pushed or pulled as the patient lifter is moved around.



Avoiding cable damage

Be sure that your operating instructions inform the user about the possible cable risks.



	 CAUTION
	The cables (particularly the connecting cable) should not be run over. In order to prevent injuries or drive damage, no mechanical strain should be placed on the cables.

7.2 Notice for operating with optional configuration**Configuration: Attached nut**

	 CAUTION
	You should never screw the stroke pipe into the drive. A stroke pipe that has been screwed into the drive can result in fatigue fractures.

	 CAUTION
	The stroke pipe may only be turned one half rotation out of the drive. Do not rotate the stroke pipe further out. An over-rotation can lead to unsafe operations. The stroke pipe clevis must not be detached from the stroke pipe!

Configuration: Mechanical reset function (clevis quick release: GQR)


	 CAUTION
	The mechanical reset function (GQR) is not a safety system and does not avert danger.

Note the following information when operating the MEGAMAT MFZ with the mechanical reset function GQR (clevis quick release):


- There should be no external dirt on the drive. Contaminants could prevent the mechanical quick release function (GQR) from functioning reliably. We therefore recommend that a visual inspection (refer to section 9.1), cleaning, and a quick-release functionality check all be included in the service plan for your end product.
- The GQR is equipped with a high-quality clutch system, which can be actuated at any time, even when under load.
- In an emergency, you can move the drive mechanically using the mechanical release. Use the mechanical release mechanism to return the drive to its home position.
- A conventional Bowden cable, which is secured in the GQR's cable holder (refer to section 6.2.3), is used for providing this mechanical release. The Bowden cable is not included in our delivery.
- The Bowden cable holder must be pulled until it reaches the end stop (all the way up) in order to ensure that the GQR functions properly. The GQR's clutch opens up when the pull force reaches 110 N. The pull force must not exceed 250 N, otherwise the GQR could be damaged.


- ▶ Depending on the implementation of your end product, it may be possible to lower the product simply using gravity. If your end product does not lower under its own weight, then you should apply gentle pressure on it to help it go down.
- After the mechanical reset function has been carried out, the clutch will engage automatically when the drive system is moved back into position. The drive system is then ready.


Make sure that your operating instructions inform the user about how to avoid damaging the GQR system:

NOTICE
<div data-bbox="284 801 363 882"></div> <div data-bbox="392 748 1417 842">The GQR system is designed to be used for the push direction of the drive. When actuating the GQR, make sure that the manual adjustment motion takes place in the pushing direction. Otherwise the GQR could be damaged.</div> <div data-bbox="392 855 1417 981">The GQR is also normally fitted with a mechanism that protects it from getting stuck. If this mechanism is actuated improperly (for example, a quick release in an unintended direction without operating the Bowden cable), then there will be increased wear and tear on the system which could result in a premature system failure.</div>

Configuration: Mechanical reset function (lifter release, LR)

	CAUTION The mechanical reset function (LR) is not a safety system and does not avert danger.
---	--

	NOTICE The MEGAMAT MFZ drive with the mechanical reset function lifter release (LR) has a limited five-year lifespan. The actual duration of use can be determined from the date of production as specified on the ratings plate (refer to section 2.5.1).
---	--

	NOTICE Avoid exerting any lateral (shear) forces on the drive. <ul style="list-style-type: none">• Do not pull or push on the guidance profile or the stroke pipe (refer to).• Never pull, push or place any load on the stroke pipe or guidance profile during transport or while they are in their no-load position.
---	--

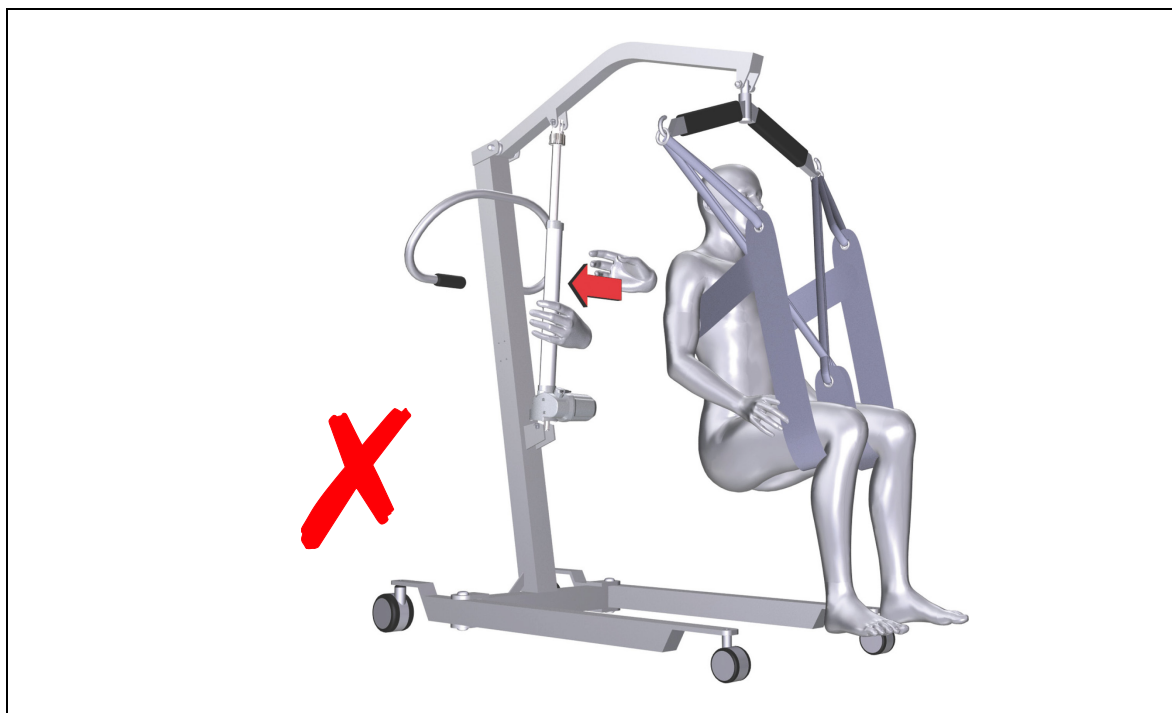


Figure 26 Avoiding lateral (shear) force

- Observe the following when operating the MEGAMAT MFZ together with the mechanical reset function LR (lifter release):
- With the patient lifter, the stroke pipe is moved in mechanically using the mechanical reset function.

To carry out the mechanical reset function, activate the safety clutch on the lifter release (refer to Figure 27):

- Pull down on the safety clutch,
- Turn the safety clutch,
- Close the safety clutch.

Press the safety clutch back to its initial position after the mechanical reset function has been carried out (refer to Figure 27). The safety clutch then engages back into position.

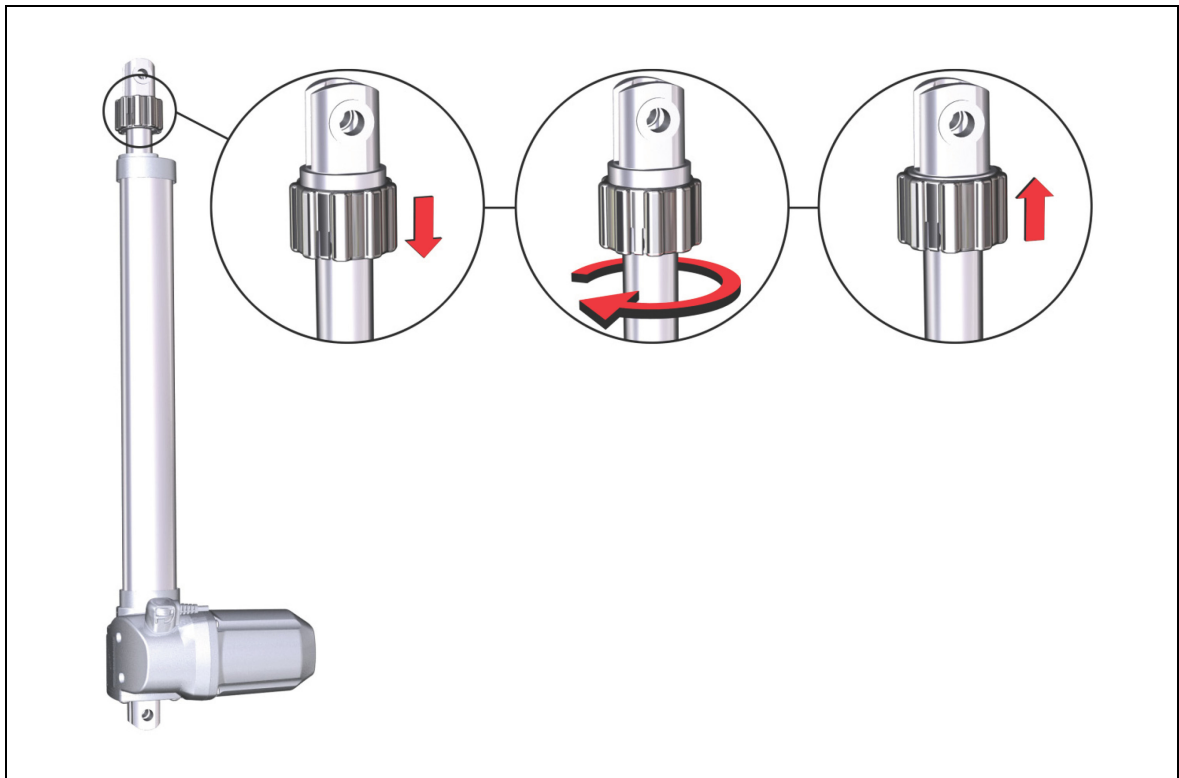




Figure 27 Operating the mechanical reset function (lifter release, LR)

8. Troubleshooting

This chapter describes troubleshooting methods for fixing problems. If you experience an error that is not listed in this table, please contact your supplier.

	 CAUTION
	Only qualified specialists who have received electrician training should carry out troubleshooting and repairs.

Problem	Possible cause	Solution
The handset or drive system is not functioning.	There is no mains supply voltage.	Connect the mains power.
	The hand switch or drive system is defective.	Please contact your supplier or sales agent.
The drive is suddenly not capable of movement.	Possibly the thermal circuit breaker on the transformer has been triggered.	The drive system should be allowed to pause for 20 to 30 minutes.
	The thermal fuse on the transformer may have been triggered.	Please contact your supplier or sales agent.
	The unit's fuse may have been triggered.	Please contact your supplier or sales agent.
	There is no mains supply voltage.	Connect the mains power.
	A lead-in connection has been interrupted (mains power, hand switch or auxiliary drive).	Check the cables and reinsert them, if required.
The motor is running but there is no motion.	The safety clutch on the mechanical reset function (lifter release, LR) is open.	Close the safety clutch for the mechanical reset function (lifter release, LR).
	The GQR Bowden cable is not in its home position. There is too much tension on the Bowden cable.	Adjust the Bowden cable.
The motor is running but there is no adjustment motion. or: The drive is resetting itself.	The override function of the drive has been used continuously for adjusting the end product.	Replace the drive. (The internal override function on the GQR is malfunctioning. It is a protective function and not suitable for making adjustments.)

Problem	Possible cause	Solution
The drive or the GQR function no longer functions after several cycles.	The mechanical end stops in your end product are missing or do not fit within the drive's installation dimensions.	Check the end stops. Do the end stops fit within the installation dimensions of the drive? Redesign the end stops if necessary.
The safety clutch of the lifter release (LR) does not close.	The teeth in the safety clutch cannot engage properly.	Twist the safety clutch a bit until the teeth engage into position.

9. Maintenance

- You should only use spare parts which have been manufactured or approved by DewertOkin. Only these parts will guarantee a sufficient level of safety.

9.1 Maintenance

Type of check	Explanation	Time interval
Check the function and safety of the electrical system.	A qualified electrician should carry out this inspection. (Refer to the "Electrical connection" section in the "Installation" Chapter.)	Periodic inspections can be carried out at intervals based on the risk assessment which you conduct for your end product.
Look over the housing periodically for any signs of damage.	Check the housing for breaks or cracks. The IP-class protection will be impaired by any breakage or cracks.	At least every six months.
Look over the plug-in connections and electrical access points for signs of damage.	Check that all electrical cables and connections are firmly seated and correctly positioned.	At least every six months.
Look over the cables for any signs of damage.	Check the connecting cables for pinching or shearing. Also check the strain relief and kink protections mechanisms, in particular after any mechanical load.	At least every six months.
Periodic functional test of the end switches.	Move the drive to the end positions in order to test the end switches.	At least every six months.
Periodic check of the mechanical reset function (lifter release, LR).	Check the mechanical reset function by opening the safety clutch (refer to the "Configuration: mechanical reset function" section found in section 7.2 "Notice for operating with optional configuration").	At least every six months.
Check the bearing bushings.	Check the mechanical reset function (lifter release, LR) of the bearing bushings in the clevis.	At least every six months.
Periodic visual inspections, cleaning, and then a mechanical-reset functionality test (clevis release, GQR).	Check to see if the mechanical reset function (GQR) is dirty. Clean and check function again.	At least every six months.

9.2 Cleaning and care

The MEGAMAT MFZ drive was designed so that it would be easy to clean. Its smooth surfaces simplify the cleaning process.

**NOTICE**

Never clean the drive in an automated washing system or with a high-pressure cleaner. Do not allow fluids to penetrate the drive. Damage to the system could result.

- 1 Always disconnect the mains power plug of the control unit or double drive (and the battery plug if present) before you start to clean the unit!
- 2 Clean the MEGAMAT MFZ drive using a moist cloth while it is in its retracted position.
- 3 Make sure that you do not damage the drive's connecting cable.

**NOTICE**

Do not use a cleanser that contains benzene, alcohol or similar solvents.

10. Disposal

10.1 Packaging material

The packaging material should be sorted into recyclable components and then disposed of in accordance with the appropriate national environmental regulations (in Germany according to the recycling law KrWG from 01.06.2012; internationally according to the EU Directive 2008/98/EC (Waste Framework Directive WFD as of 12.12.2008)).

10.2 Drive components

The MEGAMAT MFZ drive consists of electronic components, cables and metal and plastic parts. You should observe all corresponding national and regional environmental regulations when disposing of the MEGAMAT MFZ drive.

The disposal of the product is regulated in Germany by Elektro-G, internationally by the EU Directive 2012/19/EC (WEEE), or by any applicable national laws and regulations.



The MEGAMAT MFZ drive should not be disposed of with normal household waste!

Einbauerklärung

nach Anhang II der EU-Maschinenrichtlinie
2006/42/EG

Der Hersteller

Declaration of Incorporation

*According to Appendix II of the EU Machinery
Directive 2006/42/EC*

The manufacturer:

DewertOkin GmbH
Weststraße 1
32278 Kirchlegern
Deutschland - Germany

erklärt hiermit, dass nachstehend beschriebene
unvollständigen Maschinen

*declares that the incomplete machines described
below*

MEGAMAT MFZ MEGAMAT MF(n)¹⁾

die folgenden grundlegenden Anforderungen der
Richtlinie Maschinen (2006/42/EG) erfüllt:

*complies with the following basic requirements of the
Machinery Directive (2006/42/EC):*

Abschnitt:

Sections:

1.1.3; 1.3.3; 1.3.4; 1.3.7; 1.5.1; 1.5.2; 1.5.5; 1.5.6; 1.5.7; 1.5.8; 1.5.9; 1.5.10; 1.5.13

Die unvollständige Maschine darf erst dann in Be-
trieb genommen werden, wenn festgestellt wurde,
dass die Maschine, in die die unvollständige Ma-
schine eingebaut werden soll, den Bestimmungen
der Richtlinie Maschinen (2006/42/EG) entspricht.

*You may only operate this incomplete machine after
you have confirmed that the end product (into which
this machine will be installed) complies with the
Machinery Directive 2006/42/EC.*

Der Hersteller verpflichtet sich, die speziellen Unter-
lagen zur unvollständigen Maschine einzelstaatli-
chen Stellen auf begründetes Verlangen elektronisch
zu übermitteln. Die zur Maschine gehörenden
speziellen technischen Unterlagen nach Anhang VII
Teil B wurden erstellt.

*On reasonable request, the manufacturer is obliged
to send the special documentation accompanying the
partially completed machinery in electronic form to
the appropriate national institution. The special
technical documents corresponding to the machine
have been created according to Appendix VII, part B.*

Für die Zusammenstellung der technischen Unter-
lagen ist bevollmächtigt: DewertOkin GmbH
Weststraße 1
32278 Kirchlegern
Tel.: 05223 979-0
Deutschland - Germany

*For preparation of the technical documentation is
authorized:* DewertOkin GmbH
Weststraße 1
32278 Kirchlegern
Tel.: 05223 979-0
Deutschland - Germany



Kirchlegern, Germany 25 November 2019

Dr.-Ing. Josef G. Groß
Geschäftsführer / Managing Director

¹⁾ (n) steht für/stands for 1 – 5

EG-Konformitätserklärung

Nach Anhang IV der EMV-Richtlinie 2014/30/EU

Nach Anhang IV der EU-Niederspannungsrichtlinie 2014/35/EU

Nach Anhang VI der RoHS-Richtlinie 2011/65/EU (inkl. Delegierte Richtlinie (EU) 2015/863)

Der Hersteller

EU Declaration of Conformity

In compliance with Appendix IV of the EMC-Directive 2014/30/EU

In compliance with Appendix IV of the LVD-Directive 2014/35/EU

In compliance with Appendix VI of the EU RoHS Directive 2011/65/EU (incl. Commission delegated Directive (EU) 2015/863)

The manufacturer

DewertOkin GmbH
Weststraße 1
32278 Kirchlegern
Deutschland - Germany

erklärt hiermit, dass das Produkt

declares that the following product

MEGAMAT MFZ MEGAMAT MF(n)¹⁾

mit DewertOkin Steuerung / with DewertOkin Control unit

die Anforderungen folgender EG-Richtlinien erfüllt:

meets the requirements of the following EU directives:

Richtlinie über elektromagnetische Verträglichkeit 2014/30/EU

Electromagnetic Compatibility Directive 2014/30/EU

Niederspannungsrichtlinie 2014/35/EU

Low Voltage Directive 2014/35/EU

DELEGIERTE RICHTLINIE (EU) 2015/863 DER KOMMISSION vom 31. März 2015 zur Änderung von Anhang II der Richtlinie 2011/65/EU des Europäischen Parlaments und des Rates hinsichtlich der Liste der Stoffe, die Beschränkungen unterliegen.

COMMISSION DELEGATED DIRECTIVE (EU) 2015/863 of 31 March 2015 amending Annex II to Directive 2011/65/EU of the European Parliament and of the Council as regards the list of restricted substances.

Angewendete Normen

Applied standards:

- EN 60335-1:2012/A11:2014
- EN 55014-1:2006/A1:2009/A2:2011
- EN 55014-2:1997/A1:2001/A2:2008
- EN 61000-3-2:2014
- EN 61000-3-3:2013
- EN 62233:2008

Konstruktive Änderungen, die Auswirkungen auf die in der Montageanleitung angegebenen technischen Daten und den bestimmungsgemäßen Gebrauch haben, das Produkt also wesentlich verändern, machen diese Konformitätserklärung ungültig!

This declaration of conformity is no longer valid if constructional changes are made which significantly change the drive system (i.e., which influence the technical specifications found in the instructions or the intended use)!



Kirchlegern, Germany 25 November 2019

Dr.-Ing. Josef G. Groß
Geschäftsführer / Managing Director

¹⁾ (n) steht für/stands for 1 – 5

Additional information

MEGAMAT MFZ drive system

The following standards and norms were used in the versions with at least IPX4 and higher in according to

- EN 60601-1:2006 + A1:2013, IEC 60601-1:2005 + A1:2012 (short description: Edition 3.1), Medical electrical equipment.
- EN 60601-1-2:2015, IEC 60601-2-2:2014 (short description: Edition 4.0), EMC

IEC/EN60601-1, Section 4	General requirements
IEC/EN60601-1, Section 6	Classification
IEC/EN60601-1, Section 7.1	Labelling – general
IEC/EN60601-1, Section 7.2	Labelling – inscriptions
IEC/EN60601-1, Abschnitt 8	Protection against electrical danger
IEC/EN60601-1, Section 11.1	Overheating protection
IEC/EN60601-1, Section 11.2	Fire prevention
IEC/EN60601-1, Section 11.3	Design requirements for fire-resistant housing
IEC/EN60601-1, Section 13	Dangerous situations and error conditions
IEC/EN60601-1, Section 16.6	Leakage current
IEC/EN60601-1, Section 17	Electromagnetic compatibility
IEC/EN60601-2-52, Section 201.11.6.5.101	Protection against water ingress: only applied for at least IPX4
IEC/EN60601-2-52, Section 201.9.8.3.2	Static load

DewertOkin GmbH
Weststrasse 1
Kirchlengern 32278, Germany
Tel: +49 (0)5223/979-0
Fax.: +49 (0)5223/75182
<http://www.dewertokin.de>
Info@dewertokin.de

ID No.: 68004